

# Owner's Operational Guide

---

## Model Aqua-Ra

---

**IMPORTANT :** keep this guide for future reference - Do Not dispose of this guide.



**WARNING :** This table has many moving parts, incorrect care and / or use of this table may result in serious body injury, read and understand this guide before using this table.



**CAUTION:** To avoid serious body injury ensure that extreme care is taken when adjusting the tables moving parts, make certain that all body parts – yours and person/s on, near or surrounding the table are away from moving parts.



**IMPORTANT:** Ensure that power cords are kept away from moving parts – avoid crushing them when table is raised up & down. Inspect cords regularly to ensure they are well protected against damage.

Ensure that all staff / operators of the table are aware of labeling on the table and this guide. Some labels will be on the underside of table. Ensure they read and understand all information before use.

With proper care and maintenance your new table will provide years of superior, trouble-free service. This document will help familiarize you with the equipment, outline your warranty, provide cleaning and maintenance instructions, and include contact information in the event you need technical support.

### WARRANTY INFORMATION

The frame is warranted to be free from structural defect for 5 years and motors are warranted for 3 years. Other components such as the foot pedal, hand control, hydraulic cylinders, cables, and cable levers are covered by manufacturer's warranty for 12 months. All warranties apply to the original owner only; fabric and foam are not covered by warranty. Above warranty's are based upon the table being used for its intended purpose and has been operated and maintained in accordance with this guide. Table and its components must not have been subjected to misuse or unreasonable use as determined by the manufacturer.

### CARE AND MAINTENANCE

**Cleaning:** Your table **MUST** be cleaned after each use – at least once daily even if covers over vinyl are used. Perspiration, massage oils etc must be removed as they may dry and crack vinyl. The vinyl fabric on this table is a quality product. Only non-abrasive, water-based cleaning agents and foam, or mild soap & water may be used for

cleaning these fabrics. As with any vinyl, it is extremely important to pre-test your cleaning method on a hidden area of the fabric to determine if the color bleeds or if the fabric shrinks. When in doubt, seek the help of a professional cleaning service.



**IMPORTANT:**

**If table is use in wet environment and subjected to regular water spray / humidity -**

The frame and all metal parts **MUST** be wiped dry and sprayed well and wiped over thoroughly at the end of each day with a water repellent solution such as WD 40 or similar to prevent rust or corrosion of surfaces.

If table is not used in wet environment but is subjected to humid conditions or is located in coastal areas then all metal surfaces should be wiped over with clean damp cloth and sprayed with WD 40 or similar at least once a week.

If table has wood trims then the wood trim must be dried daily and wiped over thoroughly at least once a week with Norwegian wax or similar to prevent damage to wood as a result of water / humidity.

**REMEMBER – Never use acetone or harsh chemicals to clean any surface of this table!**



**CAUTION:**

**Maintenance:** In order to keep your table in top operating condition the table is to receive an inspection / tune-up twice yearly by a qualified technician. Connections may naturally loosen over time due to the various pressures this table will be exposed to during daily use. To guard against this, a routine preventive maintenance schedule should be followed. If these connections are left un tightened, injury to persons, damage to your table and/or squeaking may result.



**IMPORTANT – Self-repair or modification of your table without prior factory approval may void your warranty! Always check with our technical support department first.**

### PRECAUTIONS AFFECTING TABLE LIFE



**IMPORTANT:** The facespace **MUST** be removed from the table when it is used during shower treatments and may be subjected to water spray. The stitching around the vinyl is not waterproof.

When moving the table, **do not** lift it by grasping the upholstered pads at the head and foot sections of the table. Instead, reach below and lift the table by its frame. Failing to observe this method may potentially cause damage to the unit. Also take care not to set the table on one of the electrical cords connecting the control box to the wall and to the pedestal motor, which could sever them. Should you accidentally disconnect one of these cords, you will find that significant force will be required to reconnect it and restore use of the table. (This is designed safety feature.)

Lift the table slightly when moving it; dragging the table across the floor can damage the leveling feet. (Your table weighs approximately 100kgs; always have someone help you when moving it.)

It is unsafe for you or your clients to sit on the outermost edge of this table at the head and foot sections. Please avoid sitting or placing heavy objects on those sections when they are elevated. Do not sit on the facespace when it is inserted into the table, or “push off” against it when raising from the table: it is not designed to hold your full body weight.

### **OBTAINING TECHNICAL SUPPORT**

Should you have technical questions regarding your table or suspect you have a malfunction you should first contact the distributor or showroom which sold you the equipment. They will work with you to resolve any issues. If you are unsure of which distributor to contact or feel your concerns are not being satisfactorily addressed at that level, you may call **HydroCo 03 8770 4000**.

When calling we ask that you have this document and/or your original packing list handy so that we can verify the date of purchase. (For warranty purposes the date of purchase is considered the shipping date.) The model number of your table can be found on both documents.

**Electrical Requirements: 240Vac, 10 Amp supply for charging of battery pack located underside of table.**



**For safety reasons the Aqua-Ra wet / dry table is powered by low voltage battery pack that is located underside of the table. Battery pack is to be charged by connecting to a 240Vac, 10 Amp electrical supply at the end of each day of use or as specified in the motor manufacturers manual (read manual).**



**CAUTION: RISK OF ELECTRIC SHOCK**

**To avoid fatal or serious body injury do not use the Aqua-Ra table in a wet environment whilst it is connected directly to a 240Vac electrical supply or during the battery charging process.**

**This table MUST only be operated when located in a wet environment by the on-board battery pack.**

**The 240Vac electrical supply used to charge the battery pack MUST be protected by an Earth Leakage Circuit Breaker (ELCB) in accordance with**

**local / national electrical standards / code.**

**Table Weight:** 100kg (approx)

**Maximum Safe Working Load:** 300kg



The exclamation point in a triangle signifies an important item. Read the manual concerning the item mark or feature identified



The lightning flash in a triangle is a warning that hazardous energy is present



Protective earth terminal is signified by this mark